


How The Peezy Midstream Can Improve The Accuracy Of Your Urine Test

Designed by a doctor, Peezy Midstream improves outcomes for patients with UTI and those attending regular prenatal screening by increasing the accuracy of midstream sample collection.

It's now used by clinicians and patients in the UK and is backed by substantial published evidence that confirms it works in delivering accurate urine specimens, reducing retests and false-positives.

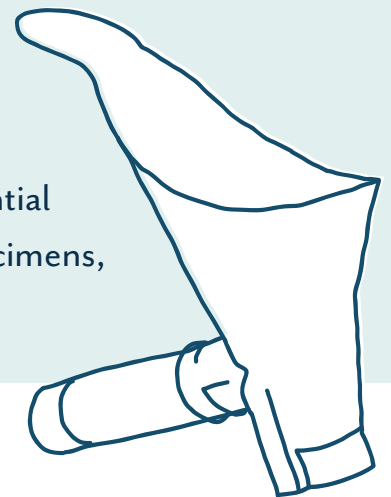

What are the problems with urine samples?

- You are usually asked to provide a “midstream” urine sample. Ideally, this means you don’t collect the first part of urine that comes out. For this to happen, you need to start-stop-start your urine stream.
- If you collect a sample from the first part of your pee by mistake, it may contain bacteria from your skin which causes contamination. This can lead to an inaccurate result and you may have to do more tests.
- On a very practical level, you are often required to give a sample in a small pot - so small it’s hard to get your pee into the pot without getting your hands wet.

How can Peezy Midstream help?

Peezy Midstream reduces false positive urine dips by 70%

Clinically proven Peezy Midstream helps with:

- Right-first-time diagnoses
- Reduced sample contamination
- Reduced false-positive results
- Infection control and patient safety
- Reduced repeat testing
- Saving time and money
- Dry hands

How does Peezy Midstream work?



1. Attach the collection tube to the Peezy Midstream device
2. Hold the device in place and start to pee as normal
3. The first part of your urine stream is automatically discarded, reducing contamination of your sample
4. The midstream of your urine is collected in the tube, without the need for you to stop and start the flow of urine
5. Any overflow is automatically discarded into the toilet, reducing the chances of spilling or cross infection
6. Remove and seal the collection tube

While all of this is happening, all you need to do is hold the device in place and pee from start to finish. The Peezy Midstream does everything else for you.

Request the Peezy Midstream for accurate urine samples. Simply print this PDF, take it to your doctor and ask them to get in touch to order. Assigned HCPCS Code A4327.

Peezy Midstream is available in the USA exclusively with Owen Mumford by contacting 1-800-421-6936 or us.info@owenmumford.com

